

Instructions for Arabian Stool




Figure 3

Top Pad (5)

Lower Pad (6)

Step 1

Arabian Stool in flat storage/delivery position. To install remove from packaging and plug into socket. Turn on.

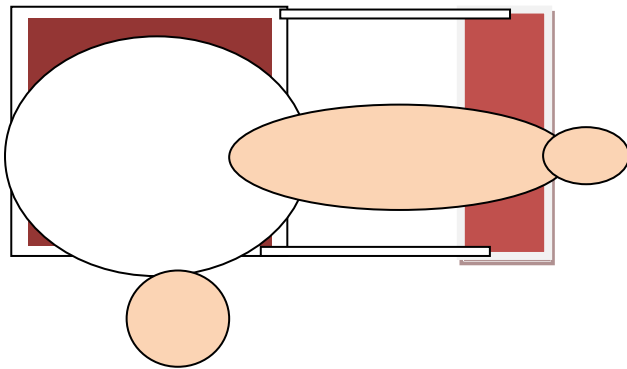

Step 2

Position client at an angle with limb to be cleaned / dressed in front of pad.

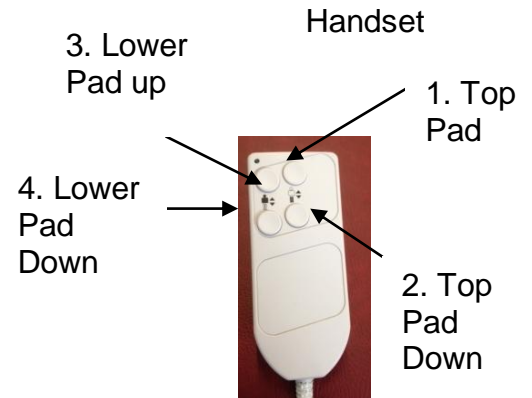


Figure 1



Figure 4

Step 3

To raise top pad (5) Press button 1 until height reached. This will maintain the position whilst the foot is cleaned / bandaged, supporting the top of the leg.



Figure 5

Step 4

To raise lower pad (6) press button 4 (this will raise lower pad 6). Until the pad is inline with the upper pad.



Figure 6

Step 5

Press button 2 (this will lower top pad 5), and support the foot whilst upper leg is cleaned and bandaged. Once bandaging is complete clean pads with steri wipes or suitable antibacterial spray (ie Dettol) return pads to vertical position Figure 3 and store.