



## **FAULT FINDING**

**Should your Baltic bed fail to function, please check the following.**

1. Check the Charger is plugged into a 13amp socket
2. Check to see if any of the following lights are showing on the charger.  
**Red - Charging**      **Amber - Charged**      **Green - Fully Charges**
3. Check the connections of handset to the bed, the handset can be fitted to either the left or right hand side of the bed.
4. Ensure the bed frame is free of obstacles, ensuring all frames can lay flat to enable connections can be made.

**Having checked the above, if the bed is still not functioning please contact**

**Centrobed Ltd on 01233 635353**


**Please could you ask for the Repairs Department**

# Baltic Handset Instructions

Each button has its own function described below. If you press button number 1 when one or both of the sides are already raised. The sides (turning) will be lowered before the back or knee section raises. The same applies to button 3 and 5, if the back or knee sections are raised then this will lower before turning begins. Anti-Trendelenburgh (feet higher than head) is not recommended unless under medical supervision.



3. Raise Knee

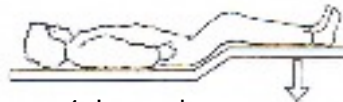

4. Lower knee



1. Raise to a sitting position



2. Lower from a sitting position



7. Raise left side




8. Lower left side



9. Raise right side




10. Lower right side

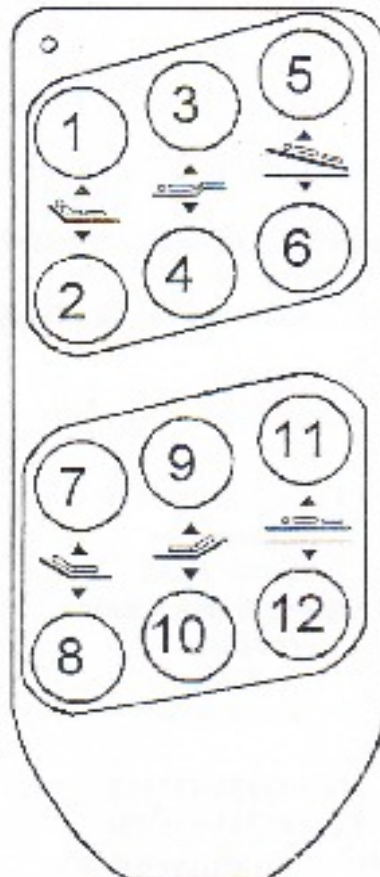


5. Positive Trendelenberg



6. Decrease Trendelenberg

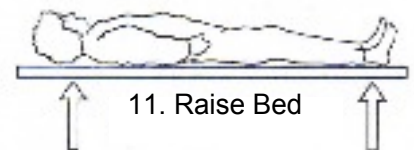

11. Raise Bed

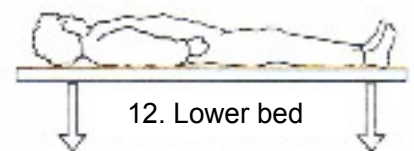

12. Lower bed