



Banda Bariatric Platform Bed

Instructions for installation and use.

Upon installation:

1. Plug in the control box but do not turn it on at the mains
2. Push down both buttons on the handset to clear any residual power in the system.
3. Hold down the down button firmly and turn on the mains
4. A clicking noise will then come from the control box indicating that the system is ready to use.
5. In the event of a power cut or the system being turned off at the mains please go back to step 1.

Centrobed Ltd, Anglo House, Wotton Road, Kingsnorth Industrial Estate, Ashford, Kent, TN23 6LN
T: 01233 635353 F: 01233 635351 E: sales@centrobed.com W: www.centrobed.com